

Quick reference instructions (Panduan Rujukan Ringkas)

Product demonstration (Produk demonstrasi)

Please scan the barcode beside to watch the product demonstration of COVID-19 RTK.

Sila imbas kod QR sebelah untuk menonton cara penggunaan COVID-19 RTK.



Video

Component (Komponen)





Pouch (test cassette and desiccant)	x 1
Swab	x 1
Tube with buffer	x 1
Biohazard bag	x 1
Instructions for use	x 1
Quick reference instructions	x 1

Test Procedure (Prosedur Ujian)



1. Insert tube into the round hole on the back of the box.
Masukkan tube dalam lubang yang disediakan di belakang kotak.


2. Unpeel the pouch and insert nasal swab into 2-3 cm up your nostril. Rotate the swab more than 5 times in both nostrils.
Buka bungkus komponen dan masukkan penyerap swab (2-3 cm) ke dalam lubang hidung. Gosok dua lubang hidung secara putaran lebih daripada 5 kali.

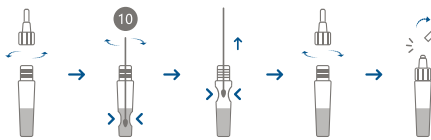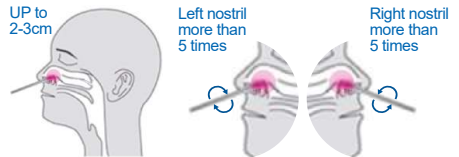

3. Immerse the swab into buffer tube. Rotate the swab 10 times and squeeze the liquid from swab before removing the swab. Screw the drip cap and break off the tip of the cap.
Masukkan hujung swab ke dalam tiub penampungan dan putar sebanyak 10 kali dalam cecair penampungan. Picit bahagian tepi tiub untuk mengeluarkan cecair dari swab sebelum swab dikeluarkan. Tutup tiub dengan penutup dan putus hujung penutup titisan.

4. Insert 3 drops into the cassette well and read result at 15 minutes.

Masukkan 3 titis ke ruang sampel atas kaset dan ambil bacaan pada masa 15 minit.

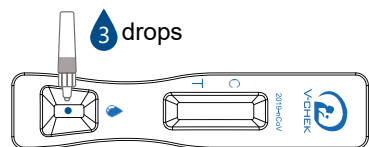




Disposal Procedure (Prosedur Perlupusan)

1. After testing, put the test cassette, swab, and sample treatment solution bottle into a biohazard bag provided and seal it tightly.
Selepas ujian, masukkan semua alatan Kit Ujian Kendiri ke dalam beg biohazard yang disediakan.
2. Dispose the bag in the waste container.
Melupuskan beg biohazard ke dalam tong sampah.



Reporting Result Method (Pelaporan Keputusan)

All results (positive, negative or invalid) have to be reported in COVID-19 Self Test section in MySejahtera app. You can also scan the QR code below for reporting.
Semua keputusan Kit Ujian Kendiri COVID-19 sama ada positif, negatif atau invalid perlu dilaporkan secara self-reporting ke dalam aplikasi MySejahtera. Anda boleh laporkan keputusan ujian melalui kod QR di bawah.

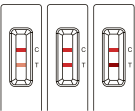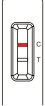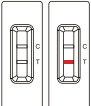Result (Keputusan)	Action (Tindakan)
<p>Positive (+)</p> 	<p>If you have obtained positive result, please proceed to private health screening facility, COVID-19 Assessment Centre (CAC) or government health clinics for further health assessment and actions. <i>Individu diminta pergi ke fasiliti kesihatan swasta, COVID-19 Assessment Centre (CAC) atau klinik kesihatan berhampiran bagi penilaian kesihatan dan tindakan seterusnya.</i></p>
<p>Negative (-)</p> 	<p>Continue to comply with all local applicable rules and protective measures. <i>Terus mematuhi semua peraturan dan langkah perlindungan tempatan yang ditetapkan.</i></p>
<p>Invalid</p> 	<p>It indicates that the operation process is incorrect or the test cassette has been damaged. In this case, please read the instruction manual carefully again and retest with a new test cassette. <i>Ini menunjukkan bahawa proses penggunaan adalah salah atau kaset ujian telah rosak. Dalam perkara ini, sila baca manual arahan dengan teliti dan buat ujian lain dengan kaset ujian sendiri yang baru.</i></p>



MySejahtera